



# INSTRUCTIONS

## **NECESSARY EQUIPMENT**

- ✓ EZ-SHAMPOO® Hair Washing Basin
- ✓ Shampoo
- ✓ Water supply or pitcher of warm water
- ✓ Wastewater collection container

## **HOW TO USE**

1. Inflate the Basin and pillow by mouth or with air pump (not included).
2. Place Basin under client's head.
3. Place end of drain hose into wastewater container.  
 The wastewater container must be placed lower than the Basin in order for the gravity fed system to work. A drain plug in the bottom of the Basin is provided for your convenience.
4. Using water supply or pitcher of warm water, shampoo hair and rinse, then towel dry as usual.

## **STORAGE & CLEANING**

Product is made of PVC and contains an anti-microbial agent. To clean, hand-wash inflated Basin using warm water and mild soap. Rinse, drain, and wipe out excess water. Leave Basin inflated for several hours after use/cleaning to allow thorough drying before deflating, folding, or storing.

 If you have concerns about allergies or how the recommended cleaning solution may affect your health, please consult with your physician or other healthcare professional before using so he/she can recommend alternatives or, product can be disposed of after each use.

CE Technical Files kept at: 3 THE TROSSACHS, CLANCY STRAND, LIMERICK, V94 83H6, IRELAND  
© EZ-ACCESS, a division of Homecare Products, Inc. *All rights reserved.*

All text and images contained in this document are proprietary and may not be shared, modified, distributed, reproduced, or reused without the express written permission of EZ-ACCESS®.